


Healthy Massage Chair
ISO9001:2000

imedics
www.imedics.com.au



ALL RIGHTS RESERVED

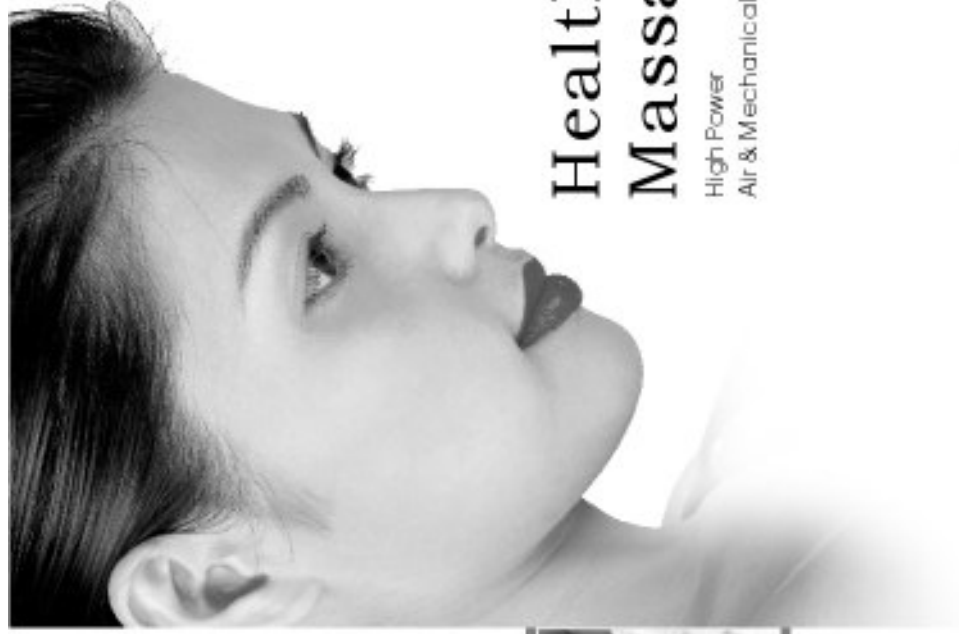

Healthy Massage Chair

High Power
Air & Mechanical Massage Combination



 You can just email us your
warranty card online
warranty eWarranty card
www.imedics.com.au

**MODEL:EXECUTIVE
USERS MANUAL**

SPECIFICATIONS

Model: EXECUTIVE
Voltage: 240V~
Frequency: 50Hz
Power: 120W
Safe Design: Class I

CONTENTS

PRODUCT DESCRIPTION	1
IMPORTANT SAFETY INSTRUCTIONS	2
FUNCTIONS AND FEATURES	6
NAME AND FUNCTION FOR EACH PART	8
ASSEMBLY METHOD	11
PREPARATIONS BEFORE USE	12
OPERATING METHOD	13
IS IT A MALFUNCTION?	15
TROUBLE SHOOTING	16
CARE AND MAINTENANCE	17
POST SERVICES	18
SPECIFICATIONS	19

TYPE EXECUTIVE

⚠ Caution:



- Before using the chair, lift the backrest pad to check for tears on the backrest centre cloth and its surrounding upholstered areas.
- Do not use the chair if the backrest centre cloth or its surrounding upholstered areas of the chair is torn, regardless of the severity of the tear(s).
- Remove the AC plug from the electrical outlet immediately and contact Medical Service centre. Continuous usage of the chair in this condition may cause injury to the user or lead to an electric shock.


TYPE EXECUTIVE

⚠ Caution:

- The following users must consult their doctor before use:
 - Those who are under medical care or are feeling unwell.
 - Those who have spinal disorders, bent or abnormal spinal conditions caused by past accident or illness.
 - Those who would like to apply the massage to an area that was or is being medically treated.
 - Those who are suffering from malignant tumours.
 - Those who are suffering from heart diseases or severe diabetes.
 - Those who are pregnant or menstruating.
 - Those who are under medical rest, or rest-cure ordered by doctors.
 - Those who have osteoporosis.
- Do not operate the chair when the AC plug is loose, the power cord is damaged, or the electrical outlet is excessively loose.
- Do not wind the AC cord or the remote control cord around the chair, or place them under the chair. Do not damage, modify or handle the AC cord, Headphone cord or the remote control cord roughly.
- Do not allow children or pets to play around the chair, i.e. behind the backrest, under the seat or footrest of the chair.
- Ensure that there is no obstruction in the paths of the backrest and the footrest during the adjustment of the recline angle of the chair.
- Do not use the chair if your body is wet. Do not operate the chair with wet fingers.
- Do not stand on the chair. Do not sit on any other part of the chair except the seat, to prevent damage to the chair.
- Please read the operation manual thoroughly for the correct methods of usage, especially the notes on safety.

IMPORTANT SAFETY INSTRUCTIONS

NOTICE

	Do not clean it with petrol, detergent and spray pesticide on it to avoid any electric shock and fire. Do not allow to sit on the backrest, armrest or footrest to avoid any hurt and damage.
	Do not move the chair on the fragile wood or carpet floor in case of any damage to the floor or truckle.
	Be sure to load not more than two people on the product to avoid any hurt or damage.
	Don't use it without clothing. It could be painful and injured with bare massage.

OPERATING METHOD

FOOTREST USING METHOD

● Footrest Ascending and Descending Method

Footrest Descending

Pull the figured handlebar and lower the footrest, when footrest reaches desired angle, release the handlebar it will be locked.



Footrest Ascending

Depart to the footrest, and pull handlebar to ascend the footrest gradually. When it reaches to desired angle, release handlebar and footrest is locked.



! WARNING When footrest is descending make sure there are no obstructions or small children near or under the footrest

OPERATING METHOD

● Backrest Ascending & Descending Method

Backrest Descending

Pull the figured handlebar and lower the backrest, when backrest reaches desired angle, release the handlebar it will be locked.



Backrest Ascending

Depart to the backrest, and pull handlebar to ascend the backrest gradually. When it reaches to desired angle, release handlebar and backrest is locked.



! WARNING When backrest is descending make sure there are no obstructions or small children behind or underneath massage chair.


FUNCTIONS AND FEATURES

Adopted Traditional Massage Modes

knead Mode: By stimulating professional masseuse massage styles, alternative knead and rub can effectively activate nerves, relieve fatigue and enhance muscular energy.




neck knead

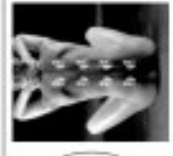


back & waist knead

Vibration: rhythmically and regularly massage your whole body.

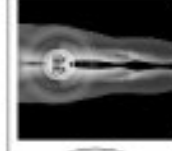

high power vibration massage

Air Pressure Massage: By inflating or deflating air into or from airbags, it can relieve muscular fatigue and pain.


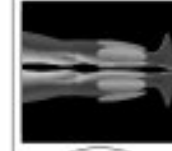


calves air pressure massage

FUNCTIONS AND FEATURES

- 1. Turn on Power**
 1. Connect power plug into socket
 2. Switch on and remote control indicator is on to enter starting mode.
- 2. Massage**

Select related massage modes by following up the remote control instructions
- 3. Turn off Power**
 1. Stop all backrest massage functions, and massage mechanism resets
 2. Shut off switch on the lower of behind chair.
 3. Detach plug from power socket

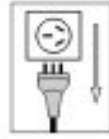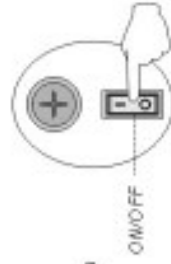

detach or uncover pillow cushion



● Use of Pillow Cushion

By using pillow cushion, it can effectively eliminate neck and shoulder massage strength. Usage of pillow cushion is optional.

! WARNING ! : Do not allow to uncover the back cushion when operating to avoid any hurt

PREPARATION BEFORE USE

- Please read the users manual carefully before use:
 - To Set up Massage Chair
- ADVICE** In order to avoid contact with surrounding objects, leave appropriate space at both ends of chair
- ADVICE** In order to eliminate possible damage to the floor, it is advisable to place firm cushioning under the chair.



● To Move Massage Chair

Lift the front of the chair by using footrest and push with two hands to start moving

! NOTICE

1. When moving chair do not have persons or objects on chair to avoid overturning
2. Do not move or drag the massage chair on a delicate floor.
3. Do not move chair by holding on to the seat or backrest, footrest must be used only
4. The chair is heavy, so handle with care



● GROUNDING CONNECTION

Make sure that all four corners connect with a flat surface

ASSEMBLY METHOD

● Assembly/Folding Method

Unfold the backrest in the direction indicated by the arrow. At approximately 120 degrees a locking sound will be heard.



Release lever situated under the backrest behind the seat, then fold backrest forward until it touches the seat cushion.



! NOTICE

To avoid possible injury when carrying out the above instructions, be sure not to put hands between footrest and backrest.

● How to Fix Back Cushion and Pillow:



SUGGESTIONS:

1. Pillow can be adjusted according to desired massage strength.
2. When massage is completed, return pillow to normal use.


NAME AND FUNCTION FOR EACH PART

Internal and External Structure

● External Structure



● Internal Structure

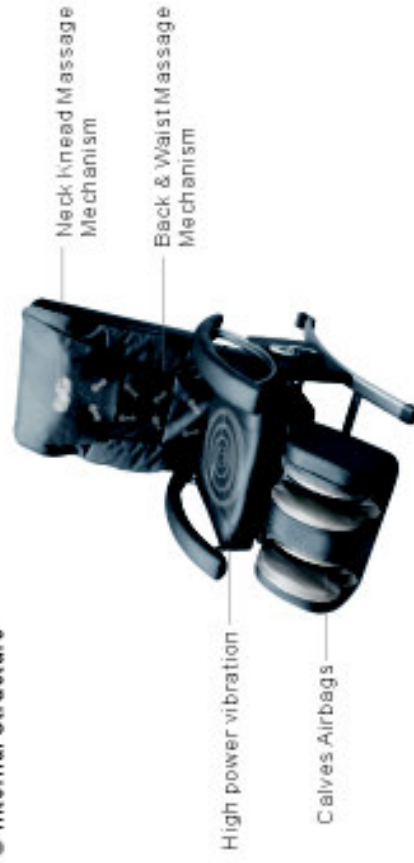


NAME AND FUNCTION FOR EACH PART

LCD



Dynamically indicates neck kneading modes
Dynamically indicates current backrest roller directions
Dynamically indicates current seat vibration modes
Dynamically indicates current air pressure massage modes

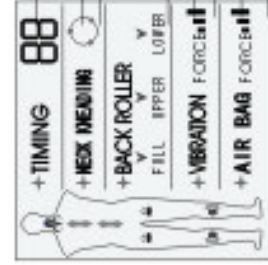

*On "reststanding mode" press "time" button to select working time, when normal working, it shows surplus time.
Dynamically indicates knead directions
Neck kneading indications
Back and waist roller indications
Vibration strength indications: "strong", "moderate", "weak"
Seat vibration indications
Airbag strength indications: "strong", "moderate", "weak"
Carries air pressure indications

NAME AND FUNCTION FOR EACH PART OF REMOTE CONTROL



- Turn on/off all massage functions. Default time is 20 minutes.
- Select message times from 10, 20 or 30 minutes
- Convert to overall automatic mode
- Start/Stop neck knead function, default mode is automation
- Alternatively select clockwise and anti-clockwise neck knead.
- Start/Stop back and waist rolling function, default mode is automation
- Alternatively select three modes from back & waist auto, backrest auto or waist auto
- Start/Stop seat vibration function, default strength is moderate force
- Select three vibration forces from weak, moderate or strong.
- Start/Stop calves air pressure function, default strength is moderate air force
- Select three air pressure strengths from weak, moderate or strong.

PRODUCT DESCRIPTION

Thanks for purchasing the massage chair, which is the wonderful combination of traditional massage modes and modern hi-tech application form. The product integrates micro-electronic control, easy-operation, and static display of LCD features and the combination of backrest movable massage mechanism, seat part multi-modes vibration mechanism and calf kneading mechanism effectively offers continuous massage to neck and shoulder part, back, waist, buttock and calf parts in varied ways.

The product can repeatedly and variedly massage the points on different parts of body for better therapy effect on the basis of Chinese medicine theory. After appropriate massage, it has been of great help in improving blood circulation, dredging whole body nerves, relieving muscle tension and enhancing metabolism.

POST SERVICES

Any other problem except for the symptoms listed on "TROUBLE SHOOTING" occurs, stop using and remove plug from the power socket, and contact sale agent or manufacturer.

● GUARANTEE (EXTRA ATTACHMENT)

Please visit www.imedics.com.au for "online" registration.

● MAINTENANCE WITHIN EXPIRY DATE

The expiry date will be one year from the date of purchasing. Please refer to local sales agent within expiry date for repair (some terms and repairs need to be paid).

● MAINTENANCE OUT OF EXPIRY DATE

Please contact to the seller for repair.

CARE AND MAINTENANCE

ADVICE Always remove plug from power socket when performing maintenance on chair. Do not remove plug from wall socket with damp cloth or wet hands to avoid accidents.

Main body

To remove dirt from plastic parts and armrests, wipe with damp cloth using mild cleaning agent and dry thoroughly with clean cloth.

Do not use alcohol, caustic agents or detergents etc. to avoid damage, discolouring, splitting etc.

Remote Control

Wipe remote control with dry cloth only.

Never use damp materials to clean remote control to avoid accidents.

Back Cushion and Pillow

If dirty, clean with damp cloth and mild cleaning agent, followed with a clean damp cloth and leave to dry.

Storing Method

After dusting and cleaning, keep chair in a dry place. If stored for long time, cover to avoid soilage.

Avoid leaving in direct sunlight, or in excessive heat as this will cause fading and colour change.

IMPORTANT SAFETY INSTRUCTIONS

KIND REMINDER




- Before use, please carefully read "IMPORTANT SAFETY INSTRUCTIONS" and use it properly.
- Please follow instructions carefully to avoid any possible personal injury or damage to the product.
- For safety reasons and in order to clearly indicate potential dangers should the product be used improperly, instructions have been marked in the following manner and should be strictly noted.

 WARNING	Once operating improperly, the user may be seriously hurt.
 NOTICE	Once operating improperly, the user may be hurt or the product damaged.

- After carefully reading, please keep this Users Manual within reach at all times.

IMPORTANT SAFETY INSTRUCTIONS

WARNING

	<p>People suffering from the following conditions should consult with their doctor before using product:</p> <ul style="list-style-type: none"> ● Cancer ● High fever ● Cardiac problems ● Emotional disorders ● Serious circulation problems ● Pregnant or menstruating women ● Acute disease ● Epilepsy ● Abnormal spinal problems ● Injuries or abnormalities <p>Use of heart pacemakers or any other internal medical electrical apparatus.</p> <p>Should for any reason, the product does not work or an abnormality occurs please disconnect power immediately and contact the distributor for an overhaul and repair.</p> <p>When descending the footrest or backrest, use moderate pressure and be sure there are no obstructions or small children near or under the footrest.</p> <p>Before use, uncover back cushion and make sure inside cushion is in good order as with all other parts. Should any part be damaged contact distributor immediately and do not use until repaired.</p> <p>To avoid fire, electrical shock or damage, the correct voltage must be used only.</p> <p>If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or similarly qualified person in order to avoid hazard.</p> <p>Keep power cord unobstructed at all times, i.e. not twisted, knotted, over flexed or pulled and do not place anything heavy on it.</p> <p>Do not allow children or people with impairments to use product without assistance.</p> <p>To avoid any possible electrical shock or injury, do not allow children to play on product.</p> <p>Do not use product in a bathroom or damp place.</p> <p>Allow only authorized technicians to handle any break-down, repair or alteration to the product.</p>
	
	

TROUBLE SHOOTING

Symptom	Please check below	Solving method
Not operating	If plug has slipped out of power socket.	Insert plug into power socket.
	If power has been switched off at chair.	Switch on power at chair.
Stops during operation even while pressing remote control	If backrest has made contact with the wall or another object.	Reposition chair, turn off power switch, wait several seconds then switch back on.
	If the chair is overloaded it will automatically turn off as a safety measure	Temporarily move from chair, switch off power at chair, wait several seconds then switch back on.
Backrest won't ascend and descend	If plug has slipped out of power socket.	Reinsert plug into power socket.
	If backrest has made contact with the wall or another object.	Move the chair into better position.
Chair Won't operate from power socket	Fuse has blown	Replace fuse.



WARNING

If above solutions haven't solved the problems, turn off power, remove plug from power socket and contact to local distributor or manufacturer for repair and maintenance.

IS IT A MALFUNCTION?

During operation, should the chair make unusual sounds or not feel right, it could be attributed to the structure and design of the chair. It most likely is a normal condition and will not influence the lifetime or mechanics of the chair.




- The clark sound when travelling up and down of the kneading mechanism
- The gears and motor sounds when in operation
- The friction of the massage chair and other materials
- Under knead, tapping and combination modes, massage speeds could be changeable due to the bearing weight on the chair
- The clark sound under tapping and combination modes
- Air bags inflating
- Air bags deflating

WARNING

 To avoid injury, fire or loss of property, do not detach, repair, alter or recondition the chair by yourself.

IMPORTANT SAFETY INSTRUCTIONS

NOTICE

	<p>While operating product, should any abnormal phenomenon occur, stop using product immediately and seek doctor's advice should it be necessary.</p> <p>After use, turn power off.</p> <p>If the supply cord is damaged, it must be replaced by the manufacturer or its service agent or similar qualified person in order to avoid hazard.</p>
	<p>Should a blackout or power failure occur, switch power off immediately and remove plug from power socket.</p> <p>When disconnecting plug from socket, be sure to hold insulating bush instead of power cord.</p> <p>When storing product, remove plug from power socket.</p> <p>To avoid electrical shock or damage, never handle plug or power socket with damp or wet hands.</p> <p>Should product receive maintenance, switch off and remove plug from power socket.</p>
	<p>Use only one product at a time, do not use in conjunction with another.</p> <p>While using the product, be sure of never falling into deep sleep or unconsciousness to avoid any possible harm or discomfort.</p> <p>At all times be sure plugs in no way obstructed with pins, dust, dirt etc.</p> <p>Do not use product while smoking or near a fire place.</p> <p>To avoid fire, do not use product on or near any heating device.</p> <p>Do not use pointed or sharp objects on or near product at any time and avoid dragging on material of product.</p>